



INSTRUCTIONS FOR USE

Description

FLEXZAN® Topical Wound Dressing is a sterile, ultra-thin, highly conformable, semi-occlusive polyurethane foam adhesive dressing which protects wounds from exogenous contamination and trauma while maintaining a moist environment conducive to optimal wound healing.

FLEXZAN® is constructed of an open cell foam with a closed-cell outer surface and a patterned adhesive coating on the patient-contact surface. Excess moisture from the wound passes readily through the adhesive pattern and is absorbed into the open cells of the foam. The closed-cell outer surface of the dressing provides a barrier to moisture, bacteria and contaminants, but has a high moisture vapor permeability, allowing evaporation of excess moisture from the foam to help prevent fluid accumulation under the dressing. FLEXZAN® has moisture management capabilities such that dry to moist wounds, even in difficult to dress areas, can be managed easily and comfortably.

Indications For Use

FLEXZAN® is intended for use in the management of:

Skin Tears

Pressure Ulcers

Venous and Diabetic Ulcers

Abrasions, Lacerations, Incisions and other Dermal Lesions

Donor Sites, Superficial Burns, and other Partial-thickness wounds

Dermatologic and Plastic Surgery Procedures

As a secondary dressing for Alginates and other Wound Fillers

Contraindications

FLEXZAN® should not be used on third degree burns or on wounds showing clinical signs of infection.

Dressing Application

1. Irrigate the wound with normal saline or water and dry the surrounding area.
2. To apply, remove the backing and place dressing over the wound as desired leaving at least an inch margin over intact skin. The dressing may be cut or overlapped to accommodate wound size or body contour.
3. Secure in place by applying gentle pressure over the dressing with your hand for approximately 20 seconds. FLEXZAN® has a "pressure sensitive" adhesive which allows the clinician or patient to easily position the dressing before securing it.

Dressing Removal

FLEXZAN® should be changed at least every 5 to 7 days, or if leakage occurs. The dressing may discolor due to the absorption of wound exudate, but need not be removed unless clinical signs of wound infection are present.

If FLEXZAN® is being used on a laser resurfaced area, it is recommended that a dressing change be performed by post-op day two.

Storage

FLEXZAN® should be stored in a cool, dry area. Sterility is guaranteed unless the product package has been opened or damaged.

How Supplied

	Flexzan® Product Number
2 inch x 3 inch	08506
4 inch x 4 inch	08518
4 inch x 8 inch	08532
8 inch x 8 inch	08564

Flexzan®
Topical Wound Dressing

10008690

Distributed by

MYLAN BERTEK
PHARMACEUTICALS INC.

Sugar Land, TX 77478
(304)285-6420